

LISTENING TO AUDIO READER ON YOUR VICTOR READER STREAM

If you own Humanware's Victor Reader Stream, you have become increasingly familiar with the tremendous capabilities Victor has to offer. One of them is being able to listen to Audio-Reader, and any other radio reading service worldwide, with a device the size of the palm of your hand.

In order to listen to Audio-Reader, you must first establish WiFi connectivity on your Victor Reader. If this is already done, you can skip this portion of the instructions and move on to the next paragraph. To connect Victor to WiFi: Power on the Victor. Once you hear "Welcome to Victor Reader", followed by a single musical note, you are ready to begin the connection process. Hit the number 7 on your Victor and listen for the "wireless" menu. Now begin tapping the number 8 on the Victor's keypad. The first thing you'll hear is "airplane mode". Make sure this is set to off. Next item when you tap number 8 is import a network connection. Hit number 8 again and you'll have the option to scan for available connections. Now press the pound key which is directly under the number 9. You will be shown several SSID options. Tap number 8 to scroll through the options. Once you've selected the option you want, press the pound key located directly under #9 and then enter your username and password using the Victor keyboard. In order to text on the Victor, number 2 = abc, #3 = def, Etc. Number 7 contains letters p, q, r, and s. Number 9 contains letters w, x, y, and z. The at key is located on the #0. The period key is located on the #1. You will need to enter your username and the password for your router.

Once you have established a WiFi connection, hit #1 on the Victor keyboard to locate Internet Radio. Initially, you will only have one playlist—the Humanware playlist. Hit #6 on the Victor keyboard to search in Ootunes. When you hear "Search on Ootunes", hit the pound key located directly below the #9 to begin your search. Use the keypad on the Victor to text the words Audio Reader and then hit pound key and Ootunes will begin searching for a match. When you locate Audio Reader, add it to your Favorites list by tapping the small square button farthest top right on the Victor keyboard. When you hear "add to favorites", hit the pound key and AudioReader will now be added. Now a second playlist has been established on your Victor—Favorites. Use #6 to navigate to this playlist whenever you want to listen to Audio Reader, and hit the play/pause button to begin listening. Hit the play/pause button again to stop the stream. The play/pause button is located at the bottom of your Victor, in between the two triangular-shaped buttons, pointing left and right respectively.

HAPPY LISTENING!